


# CROSSER<sup>®</sup> CTO Recanalization Catheter

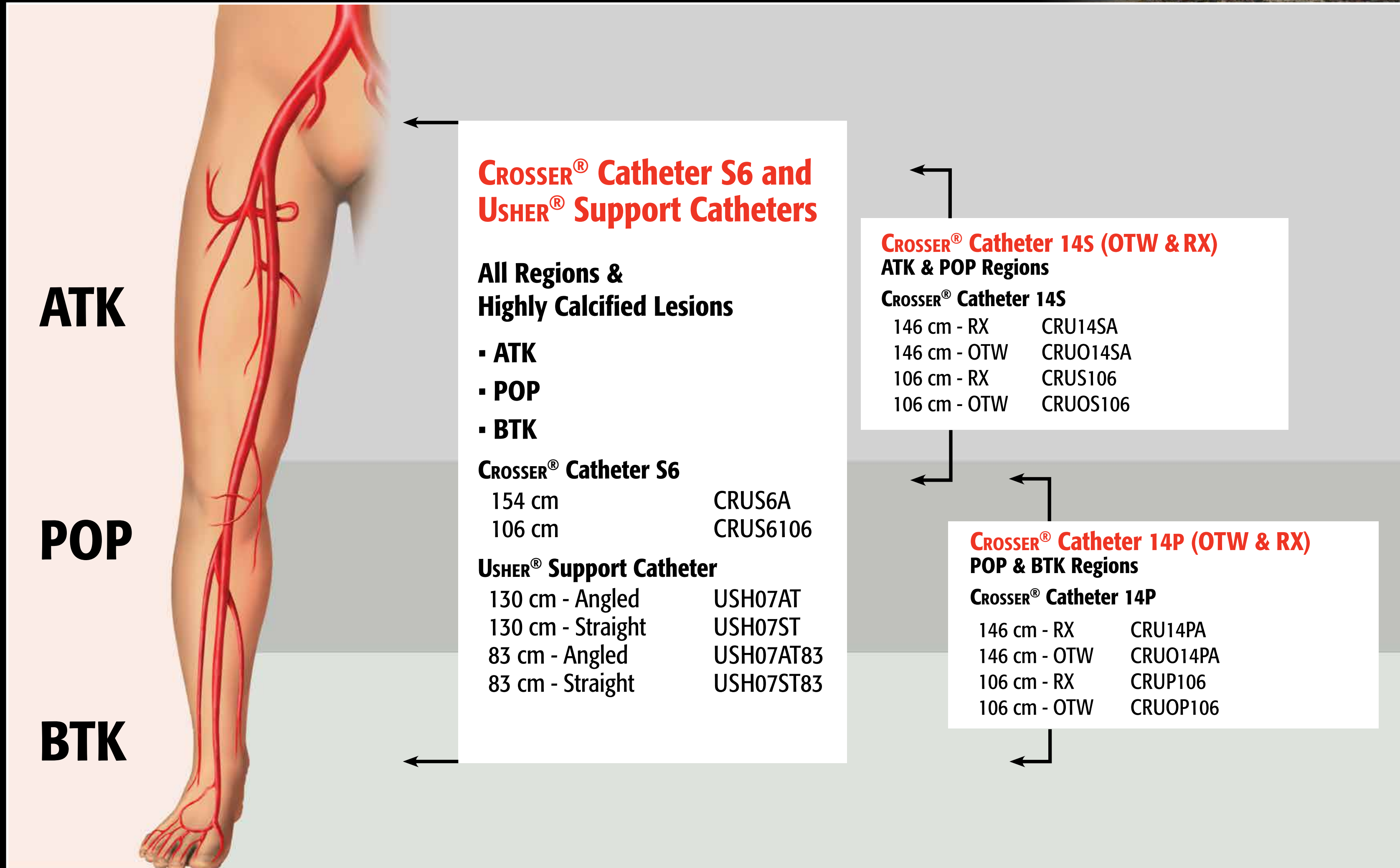

LOVE YOUR LIMBS.COM



**Indications For Use**

The CROSSER<sup>®</sup> Recanalization System is indicated to facilitate the intraluminal placement of conventional guidewires beyond peripheral artery chronic total occlusions **via atherectomy**. This device is contraindicated for use in the carotid arteries.

**Flowmate Injector Settings (pre-set)**

Flow Rate: 18 ml/min (0.3ml/sec)  
 Pressure: 200 PSI (13.8 Bar)  
 Volume: 20 ml  
 Rise/Fall: 0 seconds

**Accessories Required for Use**

Normal sterile saline - Refrigeration to 45°F (7°C) recommended

**AND**

- Compatible 150 ml Power Injection Syringe:
  - > MedRad<sup>®</sup> Part # 150-FT-Q
  - > Coeur<sup>®</sup> Part # C853-0155
  - > DeRoyal<sup>®</sup> Part # 77-400277

• 48" (122 cm) sterile infusion line

**OR**

- FLOWMATE<sup>®</sup> Procedural Kit Part# INJKIT



CROSSER<sup>®</sup> CATHETER  
**central lumen**  
 CROSSING VIA ATHERECTOMY

**Recommended Support Catheters for use with Peripheral CROSSER<sup>®</sup> Catheters 14P and 14S**

**MICROSHEATH<sup>®</sup> XL & XLT Peripheral Support Catheter**

<ul style="list-style-type: none"> <li>▪ Straight 70 cm: MSXL1770</li> <li>▪ Vertebral 70 cm: MSXL1770V</li> <li>▪ Tapered Straight 70 cm: MSXLT70</li> <li>▪ Tapered Vertebral 70 cm: MSXLT70V</li> </ul>	<ul style="list-style-type: none"> <li>▪ Straight 110 cm: MSXL17110</li> <li>▪ Vertebral 110 cm: MSXL17110V</li> <li>▪ Tapered Straight 110 cm: MSXLT110</li> <li>▪ Tapered Vertebral 110 cm: MSXLT110V</li> </ul>
------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

Bard Peripheral Vascular, Inc.  
 1625 W. 3rd Street, Tempe, AZ 85281 USA  
 Tel: 1 480 894 9515 / 1 800 321 4254 . Fax: 1 480 966 7062 / 1 800 440 5376 . www.bardpv.com  
 Please consult product labels and package inserts for indications, contraindications, hazards, warnings, cautions, and information for use.  
 Bard, Crosser, FlowMate, Love Your Limbs, MicroSheath XL, MicroSheath XLT and Usher are trademarks and/or registered trademarks of C. R. Bard, Inc. or an affiliate. All other trademarks are the property of their respective owners.  
 Copyright © 2013, C. R. Bard, Inc. All Rights Reserved.  
 S120285 Rev 0